# SCORPION

Instruments dentaires

**Notice d'utilisation**  
*Instruction for use*  
*Istruzioni per l'uso*  
*Instrucciones de empleo*  
*Bedienungsanleitung*

**Insert de détartrage dentaire**  
*Dental scaler tip*  
*Inseriti per detartraggio*  
*Insert de detartraje*  
*Scaler spitze*

**Scorpion**

4, rue Enrico Fermi  
63540 ROMAGNAT  
FRANCE  
[www.scorpion.fr](http://www.scorpion.fr)

ISO 9001 - ISO 13485

Fabriqué en France

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**Indications:** The Scorpion scaler tip is perfectly designed for teeth scaling. Due to an improved vibration, power on the scaler equipment can be reduced by the dentist from 20 up to 40 % in comparison with a classical tip. This offers a better comfort for the patient with an unchanged efficiency for the dentist. Leave the insert remove the tartar without applying pressure. The visible wear indicator, a strong but thin titanium coat, can be seen at the sharp end of the tip with its golden colour. The scaler tip is furnished non sterile. Sterilize it before use (autoclave at 134°C for 18 minutes).

**Note:** The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product.

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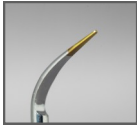
**Indications:** The Scorpion AIR scaler tip is perfectly designed for teeth scaling with the SONICflex scalers of Kavo company. For a better patient comfort, parasite vibrations are totally eliminated. The visible wear indicator, a strong but thin titanium coat, can be seen at the sharp end of the tip with its golden colour. The scaler tip is furnished non sterile. Sterilize it before use (autoclave at 134°C for 18 minutes).

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**Indications :** The EP Scorpion scaler tip is perfectly designed for teeth scaling in **Periodontology**. Its special long end will provide the perfect vibration during deep scaling within perio pockets. Set your scaler unit to **Periodontology**. Higher ultrasonic settings may fracture the tip. Use the EP Scorpion tip without any manual pressure. Replace the EP Scorpion tip when the golden-colored wear indicator is no more visible. Maintain perio pockets irrigation during treatment to avoid an excessive heating. The scaler tip is furnished non sterile. Sterilize it before use (autoclave at 134°C for 18 minutes).

**Note :** The use and the application of the product are fully achieved under the responsibility of the dental practitioner. We decline all responsibility in case of bad use of the product.

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**Indications:** The Scorpion AIR scaler tip is perfectly designed for teeth scaling with the SYNEA scalers of **W&H** company and the AIR scalers of **NSK** company. For a better patient comfort, parasite vibrations are totally eliminated. With **W&H** scalers, adjust your handpiece power to Stage 1. The visible wear indicator, a strong but thin titanium coat, can be seen at the sharp end of the tip with its golden colour. The scaler tip is furnished non sterile. Sterilize it before use (autoclave at 134°C for 18 minutes).

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